
Lynx™ MiniVib

ENGLISH: Small and elegant alerting vibrator	page 2
DANSK: Lille og elegant vibrator til varsling	side 9
DEUTSCH: Kleiner eleganter Alarm-Vibrator	Seite 16
FRANÇAIS: Vibrateur d'alerte petit et élégant.....	page 23
NEDERLANDS: Klein en elegante trilontvanger	Pag. 30
NORSK: Liten og elegant vibrator for varsling	side 37
SVENSKA: Liten och elegant vibrator för varseblivning.....	sid 44


Vibrator for the Lynx system

MiniVib forms part of Lynx – our wireless alert system for people with impaired hearing. Lynx uses various types of transmitters and receivers that alert you to noises such as a ringing telephone or doorbell, or the first signs of a fire.

MiniVib is a small receiver which sends out an alert via various vibrations and different coloured LEDs if one of the transmitters in the system is activated. The transmitters send signals wirelessly to MiniVib. One or more repeaters may also be installed in order to amplify the alarm signal and thus extend its range.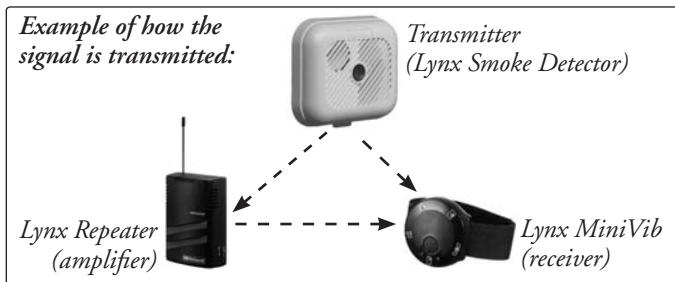


Overview

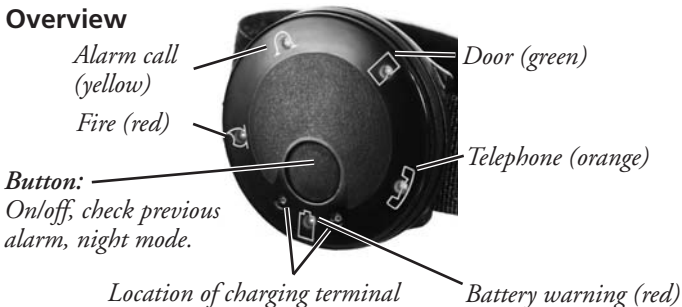Charging MiniVib

You must charge MiniVib before you can use it. It takes approx. 180 minutes to charge it fully. The battery will then last approx. four days. Charge the battery fully the first time to obtain the maximum battery service life.


Insert the charger: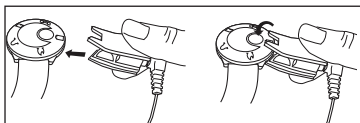

Connect the charger to the mains.

Red light comes on = charging in progress.

Disconnect the charger:

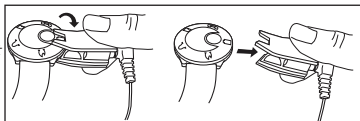Red light flashes = charging finished.

MiniVib on/off

MiniVib is on when supplied (but not charged).

Switching off

Hold down the button for about 6 seconds until the lamp for the previous alarm goes out. MiniVib vibrates briefly.

Switching on

Hold down the button for about 1 second until the lamp for the previous alarm comes on. MiniVib vibrates briefly.

Checking whether MiniVib is switched on.

Press the button briefly. When MiniVib is switched on, the previous alarm comes on and it vibrates once.

How to wear MiniVib

You can, for example, wear MiniVib on your arm or around your wrist. Check that you can feel the vibrations.

Installation and function test

1. Start by installing the transmitters that are to be used. **Read the user manual for the transmitters that you are installing!**
2. Test the system by activating the transmitters in the system one at a time. MiniVib starts to vibrate and the alarm lamp lights up. Check that the correct alarm is indicated (see below).

Alarm indication

MiniVib can receive four different types of alarm. MiniVib vibrates according to a certain pattern and the lamp for the relevant alarm comes on. The vibration is repeated once.

The Alarm Call alarm can be replaced by another alarm, e.g. by that from the Lynx Baby Alarm. **Read the user manual for the transmitters that you are installing!**

<i>Alarm</i>	<i>LED</i>	<i>Vibration</i>
<i>Fire</i>	 (red)	
<i>Alarm call</i>	 (yellow)	
<i>Door</i>	 (green)	
<i>Telephone</i>	 (orange)	

Check the previous alarm

Press the button briefly. The previous alarm comes on and vibrates once.

NOTE: If you hold down the button for more than 6 seconds, you will switch off Mini-Vib.

Battery warning

MiniVib's battery

When the battery power in MiniVib starts to fail, the ☐ lamp starts to flash while MiniVib vibrates briefly. You now have approximately one day's operation left. When the battery is almost completely flat, MiniVib will vibrate again. Charge the battery at once (see page 3)!

Transmitter batteries

When the battery power in the transmitter sending the alarm starts to fail, the ☐ symbol lights up along with the transmitter's alarm.

Smoke detector battery

A battery warning is sent to MiniVib every 8 hours. The ☐ and ∩ lamps then flash. Switch off the alarm by pressing the button and change the smoke detector's battery at once.

Night mode

If you want to switch off the alert from all alarms apart from the fire alarm and alarm call, set MiniVib to the night mode. In the night mode MiniVib also vibrates once when the battery is almost completely flat.

Setting the night mode

1. Give *three* short presses in rapid succession on the button. MiniVib vibrates briefly. The yellow lamp ∩ comes on and then flashes.

Returning to the normal mode

1. Press the button once. MiniVib vibrates briefly.

Changing channels

The transmitters in the Lynx system transmit the signal wirelessly to MiniVib. They can transmit on a total of 16 different channels (0-9, A-F). MiniVib and all transmitters must be set to the same channel so that they can communicate. They are pre-set to channel 9, and then operate together without alteration.

In some cases, it may be necessary to change the channel setting. If two or more users live near one another, the alarms can interfere. This can be rectified by changing the channel settings so that they are different for everyone. Any channel can be selected except Channel 0. (Channel 0 is reserved for a common alert such as a fire alarm.)

Channel	LED			
0	-	-	-	-
1	orange	-	-	-
2	-	green	-	-
3	orange	green	-	-
4	-	-	yellow	-
5	orange	-	yellow	-
6	-	green	yellow	-
7	orange	green	yellow	-
8	-	-	-	red
9	orange	-	-	red
A	-	green	-	red
B	orange	green	-	red
C	-	-	yellow	red
D	orange	-	yellow	red
E	-	green	yellow	red
F	orange	green	yellow	red

If you need to change the channel on MiniVib, proceed as follows:

1. Connect the charging terminal to an electric socket. In order to access programming mode: hold down the button while inserting and removing the charging terminal. Then release the button. Various LEDs flash, depending on which channel is set.
2. Change channels by pressing the button the required number of times. You step up through the channels with each press of the button.
3. To save the channel setting, wait for 15 seconds until all the lamps have gone out.

Technical data

Frequency:	433.92 MHz AM.
Number of channels:	15, and a common channel 0.
Battery:	Chargeable NiMH, Varta V 40 H. Use only this type of battery. Charging time approx 180 minutes. Operation: approx. 4 days per charging. Battery life at least one year. To change the battery, contact the place of purchase.
Mains adapter:	Make: STH. Type: P-001B.
Strap:	Elastic strap with large adjustment range. Compatible with normal watchstrap (18 mm). When changing the strap: release the pin with a needle.
Other:	Wash-proof.
Dimensions:	Diameter 38 mm, thickness 12 mm.
Weight:	12 g.

Troubleshooting

Type of fault	Action
MiniVib is not working.	<ul style="list-style-type: none"> • Check that MiniVib is on and that the battery is charged (see p 3). • Check that the transmitters are working (see transmitter user manual). • Check that MiniVib and the transmitters are set to the same channel (see p 6). • Check that Night mode is not switched on (see p 5). • The distance from the transmitter may be too great. Move the transmitter nearer or extend the system with one or more repeaters (see p 2). • If the Alarm Call is replaced by another alarm, e.g. Lynx Baby Alarm, check that the transmitter alarm symbol is set to alarm call (see transmitter user manual).
The operating time is less than two days.	<ul style="list-style-type: none"> • Contact the place of purchase to exchange the battery.

Cleaning and maintenance

Use a damp cloth to wipe MiniVib on the outside. Never use cleaning products or solvents. From time to time check that the cables and connections are intact.

If there is any leakage from the battery: scrap the battery and clean the battery plates (positive and negative) on MiniVib with a cotton bud and a little alcohol and carefully scrape off any residue from other surfaces with the end of a matchstick.

Repairs

If your product malfunctions, it must be repaired by a qualified technician. Do not attempt to open the case of the device since this would invalidate the warranty. If your product requires service, please contact your hearing care professional for assistance.

Waste disposal


This product contains electrical or electronic equipment and should be disposed of carefully in the interests of your safety and the environment. Please contact your local hearing care professional concerning disposal of the product. Used batteries should be taken to the nearest recycling point in your district.

General precautions

- Different types of radiation, e.g. from NMR or CT scanners, may damage the device. Therefore, do not wear the device in the vicinity of this type of equipment. Other types of radiation (burglar alarms, room surveillance systems, radio equipment, mobile telephones, etc) will not damage the device.
- Warning: External devices connected to the product must be safe according to the requirements of EN 60335-1.
- Only use rechargeable batteries.

Changing the battery

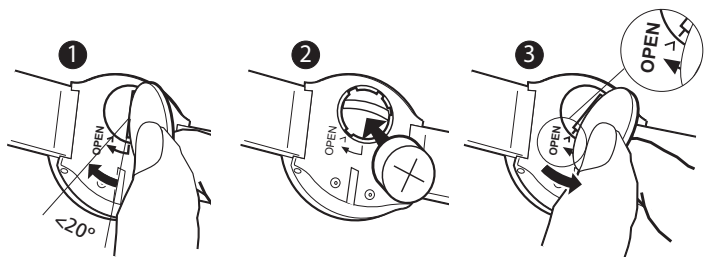CE Any issues relating to the EU Medical Device Directive 93/42/EEC should be directed to GN ReSound A/S.

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Sicom A/S
GN ReSound AB