Order number: Battery replacing kit MiniVib 200 21 99

Manual - Addendum  
English • Dansk • Nederlands • Norsk • Svenska

## Replacing the battery - MiniVib

- Open the battery cover and discard it. Note that it opens 20° *clockwise* (see Fig. 1).
- Remove the used battery and take it to a recycling point. Discard the O-ring mounted on the battery.
- Remove the O-ring from the MiniVib (see Fig. 2). To optimise its water resistance, remove any silicon residues (white) from the MiniVib with the aid of a pair of tweezers. (See Fig. 2). Replace the O-ring in the Mini-Vib.
- Slip the gasket onto the battery. Its grooved end should line up with the positive end of the battery (See Fig. 3).
- Place the battery carefully into the battery compartment so that it lies level (See Fig. 4). As soon as you slot the battery cover in place, the battery will rest in its correct position. Note! Do *not* press the battery down into the battery well, as the O-ring will then separate from the battery.
- Place the battery cover provided so that its slit points to the "OPEN" position on the MiniVib and then press it down (See Fig. 5).
- Carefully turn the battery cover to the position shown in Fig. 1.

*NB! Only use rechargeable NiMH batteries!*

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MANUFACTURER  
*in the sense of EU Medical  
 Device Directive 93/42/EEC:*  
**GN ReSound A/S**  
 Lautrupbjerg 7  
 DK-2750 Ballerup  
 Denmark

*Service and product-related  
 questions: see contact details  
 for each country or "International  
 inquiries".*

NORGE:  
**GN ReSound AS**  
 Akersgaten 45  
 Postboks 132 Sentrum  
 N-0102 Oslo  
 Telefon +47 22 47 75 30  
 Telefaks +47 22 47 75 50  
 info@gnresound.no  
 www.gnresound.no

INTERNATIONAL  
 INQUIRIES:  
**GN ReSound AB**  
**ALD Division**  
 Svarvarg. 9, 2 tr  
 SE-112 49 Stockholm  
 Sweden  
 Tel.: +46 8 441 16 70  
 Fax: +46 8 441 16 80  
 info.ald@gnresound.se  
 www.gnresound-ald.com